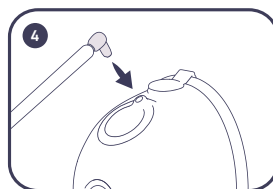
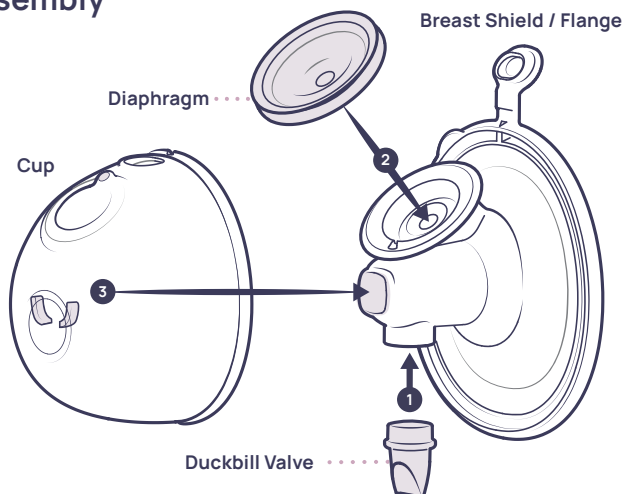
amara

go-cups

quick-start guide


Assembly

Tubing
Firmly connect the tubing to the tubing port on the cup.

For proper pump connection and use of silicone connector, please refer to the pump compatibility instructions >>>

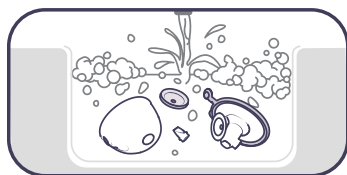
Wash


Before first use and after every use, disassemble all cup parts, wash, and boil in water for 10 minutes, ensuring full submersion of parts.

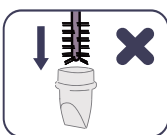Wash duckbill valve in warm, soapy water, gently rubbing tip. Pinch valve opening closed to straighten it.

Sanitize

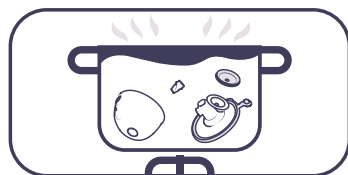Inspect valve for tears or warping. If damaged or unable to close, replace.

NOTE

- Do not wash or boil tubing. If it becomes visibly dirty, wipe clean with warm, damp cloth.
- Before use, disassemble, wash, and sanitize all parts that may have come into contact with breast or breast milk.
- Ensure all parts have been thoroughly cleaned and are completely dry before use.
- For more detailed use and care instructions, please refer to the User Manual.

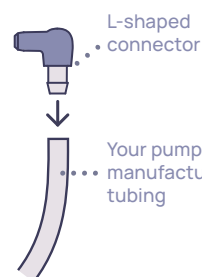
Compatibility With Your Pump*

Installation A: No silicone connector required

Spectra **Freemie** **Ardo** **Unimom** **Ameda**


Zomee **Motif** DUO, Luna

- 1 Remove the connector from your pump manufacturer's tubing.
- 2 Place L-shaped connector over your pump manufacturer's tubing.


Lansinoh

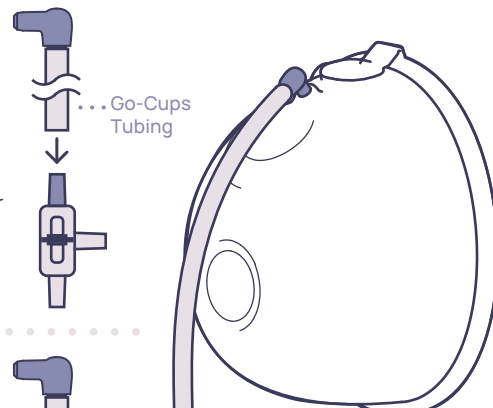- 1 Remove the silicone tubing from your pump tubing Y connector.
- 2 Place Go-Cups tubing over your pump tubing Y connector.

Motif Twist

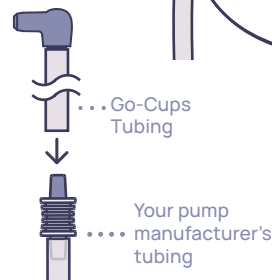- 1 Remove the silicone tubing from your pump manufacturer's connector.
- 2 Place Go-Cups tubing over your pump manufacturer's tubing.


Installation B: Use silicone connector

Medela Freestyle Flex, Pump in Style, Swing Maxi, Sonata

Baby Buddha **Pumpables** Genie Advanced

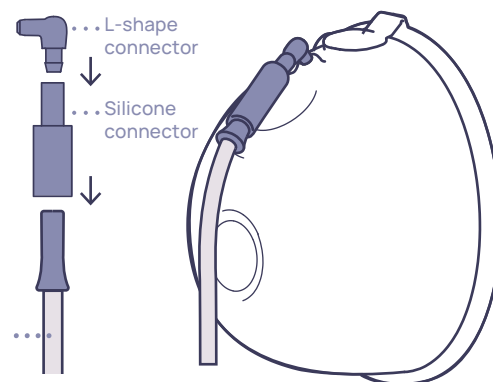- 1 Insert L-shaped connector into silicone connector.
- 2 Place silicone connector over your pump manufacturer's tubing.

*The desired suction level may vary from your pump collection kits

Scan here for User Manual.

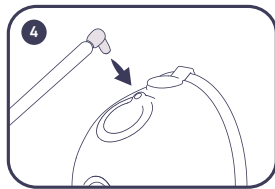
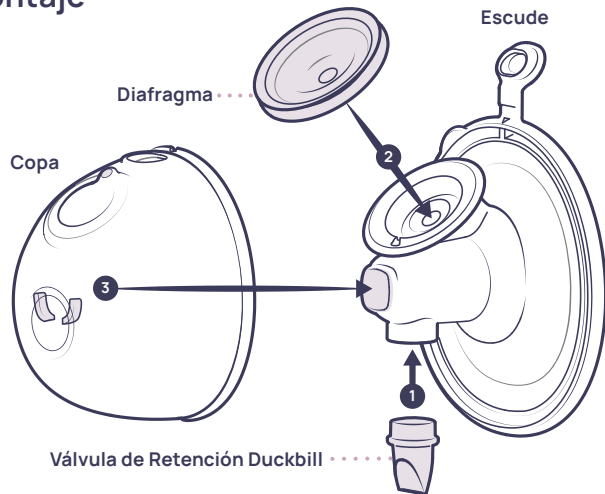
Scan here for cups compatibility and instructions.

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