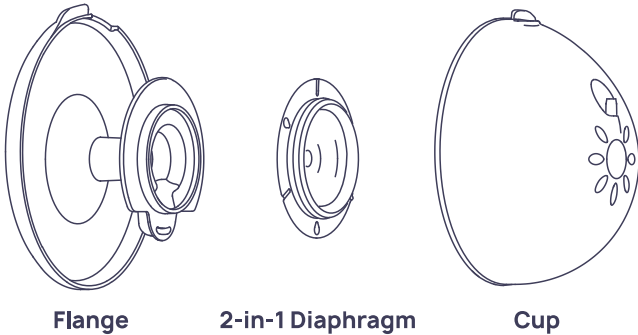
amaris

slim-cups


quick-start guide



Assembly

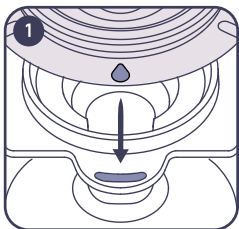
Your Amaris Slim-Cups feature a unique diaphragm designed for up to three replacement cycles before needing a full replacement. If you notice reduced suction or lower milk output, simply rotate the diaphragm to the next numbered droplet or replace it.

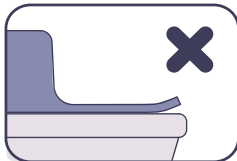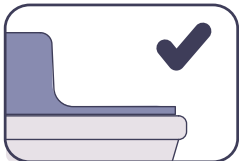
Align the numbered droplet on the diaphragm to the valve outlet on the flange. Secure by pressing around the edges of the inner circle of the flange.

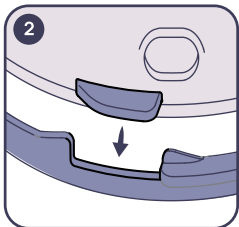
Note: Make sure the diaphragm is flat against the flange and not lifted to promote strong suction.

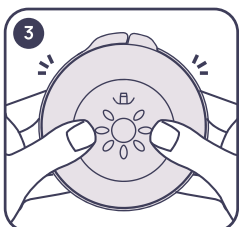Next, take the collection cup and align the tab to the open notch located on the flange and secure.

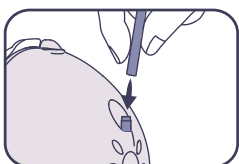Use both hands to tightly reinforce the flange and cup, applying pressure with your palms around the outer edge, ensuring a snug fit to prevent leakage.

Tubing
For proper tubing connection and use of the silicone connector, please refer to the pump compatibility instructions > >

Wash

Wash before first use and after use.

Sanitize

Sanitize before first use and once per day after use.

Note

- Do NOT wash or boil the tubing. If it becomes visibly dirty, wipe it clean with a warm, damp cloth.
- Before use, wash and sanitize all parts that may come in contact with the breast or breast milk.
- For best results, pump parts should be completely dry before use.
- For more detailed use and care instructions, please refer to the User Manual.

Please refer to the online User Manual for details

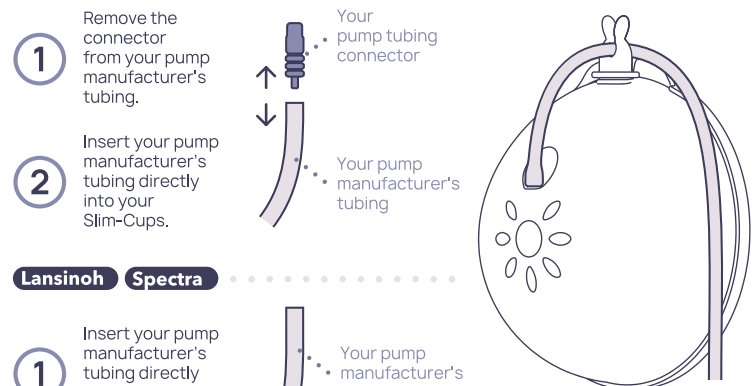
Scan code to watch assembly tutorial

Compatibility with your pump

Installation A: No Silicone Connector Required

Baby Buddha **Freemie** **Unimom** **Zomee** **Motif** DUO, Luna


Pumpables Genie Advanced, Genie Plus


Lansinoh **Spectra**

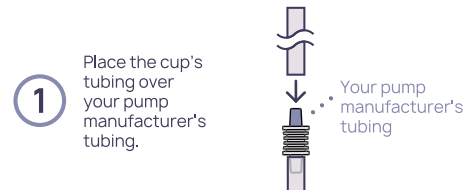Motif Twist

Ardo **Ameda** Mya Joy

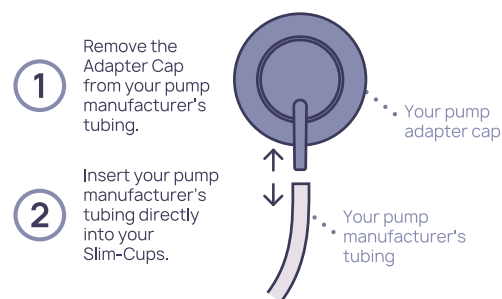
Installation B: Silicone Connector Required

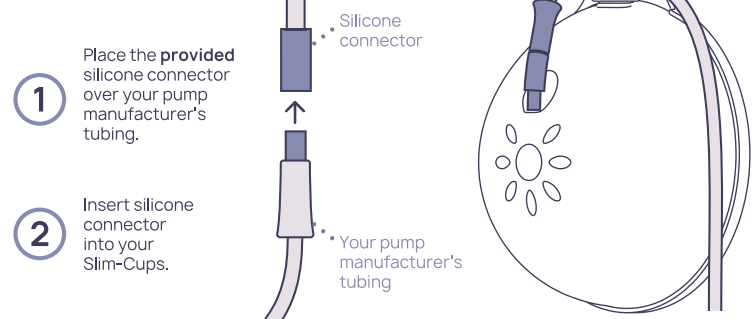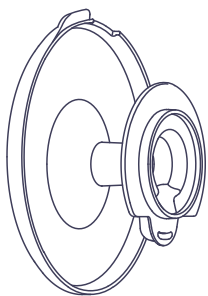
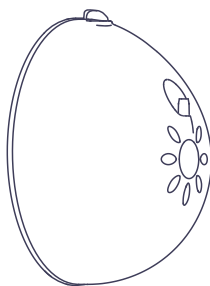
Medela Freestyle Flex, Pump in Style (slimeinate MF), Swing Maxi, Sonata

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